


Stool Specimen Collection Containers



Green top used for Ova and Parasite testing, or Fecal Leukocyte testing. Make note of the Fill line. Overfilling or underfilling may result in specimen rejection.



Orange top used for BioFire GI Panel testing in the lab. Used for H pylori send-out tests. Make note of the Fill line. Overfilling or underfilling may result in specimen rejection.

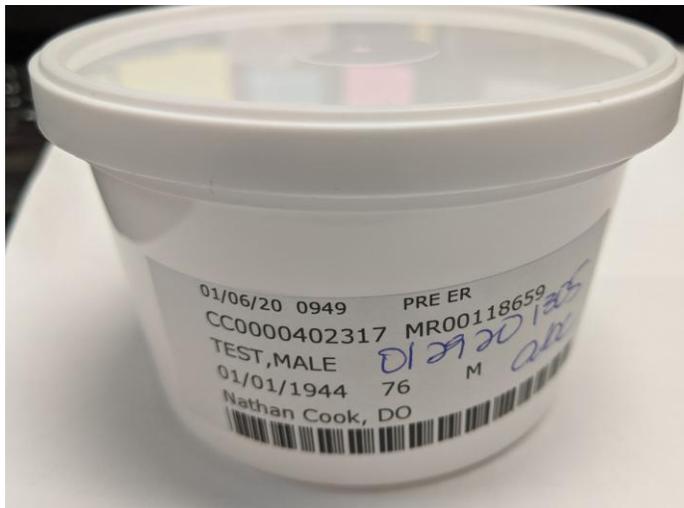

Clean container (clean, not sterile). Used for fresh stool specimens for *C diff* toxin testing and any other testing requiring non-preserved specimen. Do not use for frozen specimens.




Stool culture swab with transport media.



Collection container used for fecal occult blood testing. When collecting specimen, it is important to unscrew the lid and use the attached wand to collect sample. Do NOT scoop specimen through the top.



Container used for non-preserved specimens to be sent frozen to Mayo. Each test needs a separate container, i.e., Fecal Fat, Calprotectin, and Pancreatic Elastase would require three (3) separate containers.

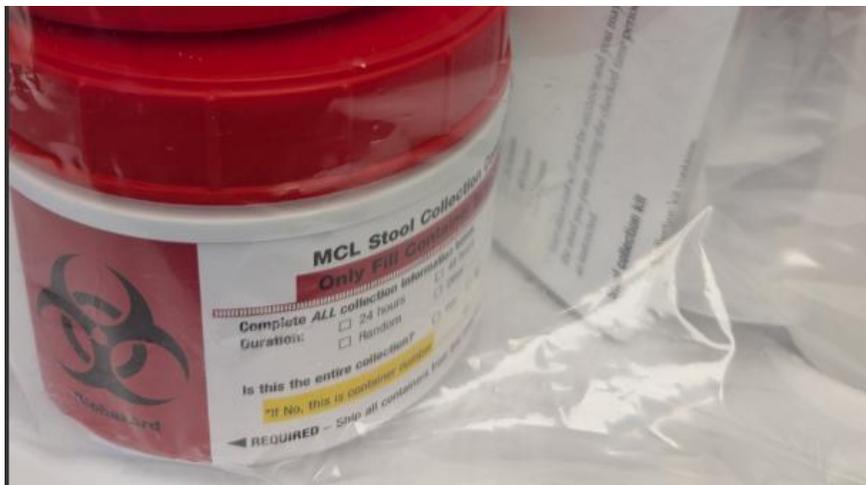

Large containers (set of 2) used for timed stool tests, i.e., 24-hour, 48-hour, or 72-hour collections.