


The Right Swab for the Job

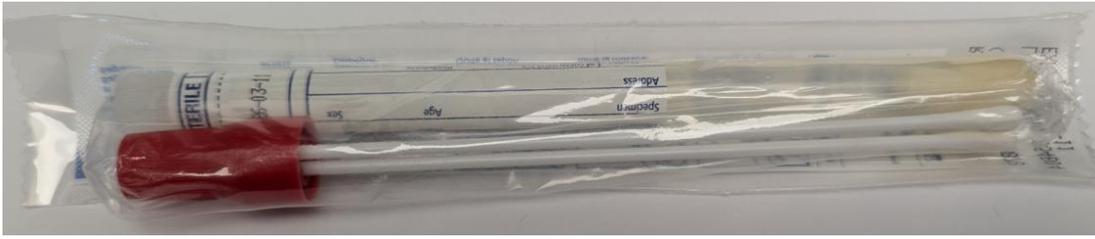

Red top used for aerobic cultures, fungal cultures, GBS testing, MRSA testing




White top eSwab used for anaerobic testing




Blue top eSwab with flexible swab used for aerobic specimens collected by ENT's



Sterile container used for other types of aerobic specimens that can't be collected with swab (such as sputum, IUD, etc.)

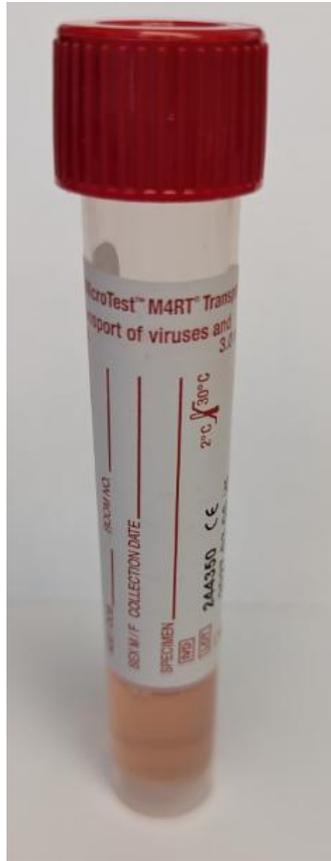

Red top Universal Viral Transport media (VTM) with swab broken at scored line. Used for BioFire Respiratory Panel, SARS/Flu/RSV PCR, HSV PCR testing, specimens intended for viral cultures/PCR testing.

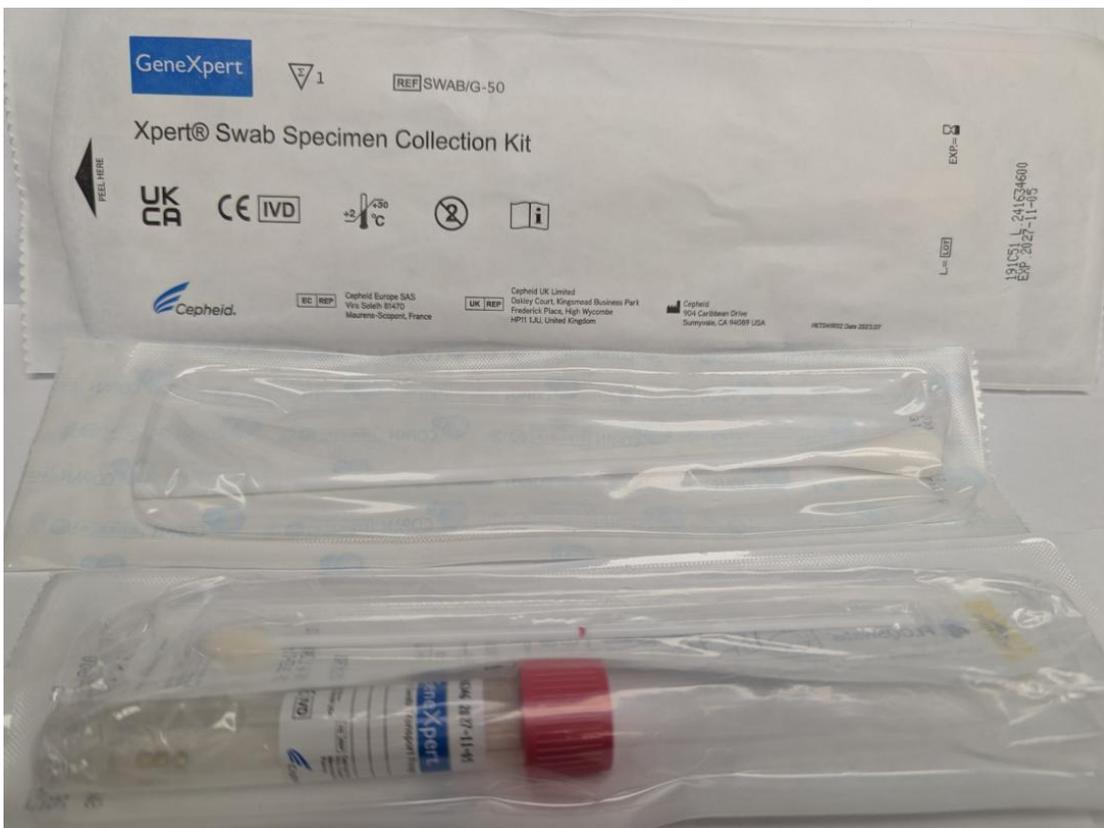

Pink top transport tube with swab broken at scored line.

Used for Vaginitis Panel (VP) and GC/Chlamydia (CTNG) testing. If testing for both, send two collection containers.

When used for CTNG testing, must be labeled with source – anal, oral or vaginal.



Swab used for Rapid Strep and back-up Strep culture. Separate swabs needed for each test.



Rapid COVID/Flu or RSV swab – Sterile Foam Tipped Applicator; Rayon swabs will be rejected.

If ordering both COVID/Flu and RSV, two swabs are needed.



Nasopharyngeal swab. Place swab in VTM prior to sending to lab.



Stool culture swab with transport media.